

CHANGE INTERVAL

Recommendation for Changing UROMED Urine Bags

Urine bag **without drip chamber**

Leg Bag


e.g. UROMED CYSTOBAG MINI 750 S Urine Leg Bag (REF 4893)

change all:

monday

wednesday

friday

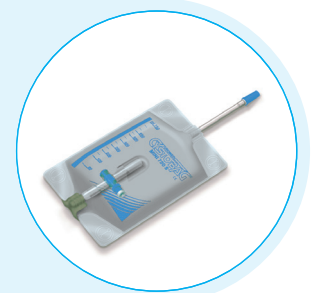

Bed Bag

e.g. UROMED CYSTOBAG LS 2000 Urine Drainage System (REF 4801)

change all:

monday

wednesday

friday



Urine bag **with drip chamber**

Bed Bag

e.g. UROMED CYSTOBAG TK 2000 »comfort«
Closed Urine Drainage System (REF 4802)

change all:

10.

20.

30.

of the respective month



**Please do not hesitate to contact us
if you have any questions.**

UROMED Kurt Drews KG

Meessen 7/11
22113 Oststeinbek
GERMANY

Phone +49 (0) 40 71 30 07-33

Fax +49 (0) 40 71 30 07-99

E-mail homecare@uromed.de

www.uromed.eu



Please find more languages on our website
www.uromed.eu under the heading "Service".